# RIDDER | *Assistent*

## Reisesauggriff

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## Travel suction grip

User manual

---

|    |    |
|----|----|
| DE | 03 |
| GB | 05 |
| FR | 07 |
| ES | 09 |
| IT | 11 |
| FI | 13 |
| SE | 15 |
| PL | 17 |
| LT | 19 |
| HU | 21 |
| SI | 23 |
| EE | 25 |
| CZ | 27 |
| SK | 29 |
| PT | 31 |
| DK | 33 |
| NL | 35 |
| HR | 37 |
| RS | 39 |
| RO | 41 |
| BG | 43 |
| GR | 45 |
| LV | 47 |

---

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## Mounting and user manual for Travel suction grip A051201-00

Dear customer,  
thank you for choosing our product. Please read the safety instructions carefully before use and keep the manual for further use. The manufacturer assumes no liability for faulty assembly or improper use of the product.

### Safety instructions:

#### **NOTE!**

##### **Intended Use:**

This suction grip serves as grab handle and supports you whilst standing up to compensate balance issues. Recommended useful life: 5 years. This grab handle is only made for temporary, recurring use and must be reinstalled before every use. These suction grab bars are suitable for humans with a max. body weight of 100 kg and serve as grab handle or support whilst standing up. Do not use it to hang things on it. Keep packaging materials away from children. Moist surfaces can impair the anti-slip function.

##### **Use:**

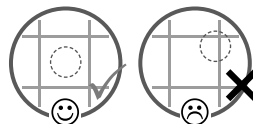
#### **WARNING!**

##### **Risk of falling at mounting place:**

The suction grab bar achieves a strong holding power due to the construction of the big suction cups and the arising vacuum by tighten them. Strong holding power only gets achieved, when the mounting position is solid enough, to hold the load and when it is guaranteed that vacuum can be build.

The mounting position must fullfill the following criteria:

- **Stability of the underground (1,5 x max. load)**
- **Cleanliness (free of grease, dirt and dust)**
- **Air impermeability (e.g. glass, glazed tiles)**
- **Smooth surface**

##### **Warning note for vacuum:**

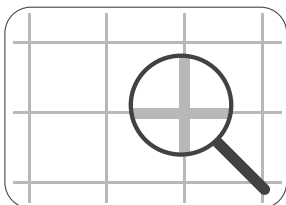
Already light and partly not visible damage of the surface, can affect the vacuum and therefore the suction power extremely. For this reason we recommend to test very carefully every desired holding position before use and to take the advise of an expertise into account.

#### **IMPORTANT! !**


**Check holding power carefully before every use! ! !**

**Do not use if the product is damaged! ! !**


## Mounting:

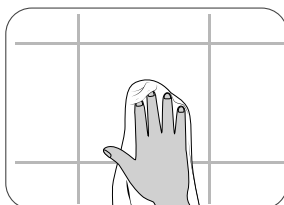
1. Check the mounting place carefully, if it is suitable and to exclude possible damages, like described previously.

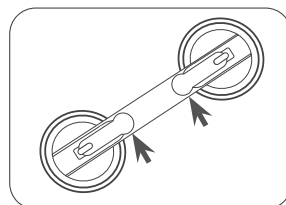
2. Clean the mounting place carefully with standard commercial cleaning agents and take care that the mounting place is completely free from dirt, dust and grease.

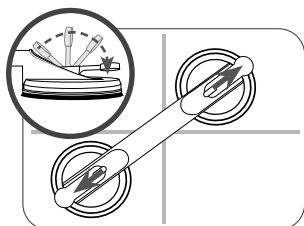3. Make sure that the fixing levers are open, before mounting.

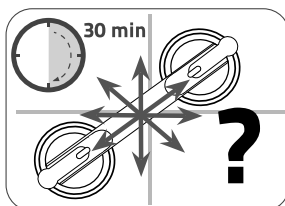
4. Press the grab handle with pressure at the mounting position on the wall and close the fixing lever. Hereby the vacuum arises.

5. Before using and therefore loading the product, please wait 30 minutes precautionary and then check the holding power again.

## Disassembly and reuseability:

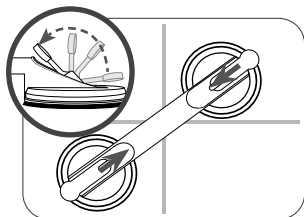6. For disassembly open the fixing levers and take the grab handle away from the wall.

7. Possible residues of dirt, or similar on the suction cup can be removed with water and standard commercial soapy water. Do not use solvent-based, acidic or abrasive cleaning agents! After proper cleansing, the grab handle is ready for use again. Please respect steps 1. - 5.

## **i** NOTE!

### Cleaning and care instructions:

Please pay attention to the cleaning and care instructions. In the case of improper use the product can be damaged. Clean it carefully with water and standard commercial detergents. Subsequently, you can dry the product with a cloth.

Never use abrasive, solvent or acidic cleaning agents to clean the vacuum suction cup. The suction cup should be cleaned with soapy water under clear water. Make sure that there is no fluff on the suction cup and the wall surface after drying, otherwise the holding force may be impaired.

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**RIDDER GmbH**